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EN: Instructions

Introduction:

The ZenSetiv non-sterile silicone plaster with split is a self-adhesive dressing which can be used around a surgically made, temporary or permanent, body opening (called a stoma), where covering is required. The stoma may be used to pass devices into the body, such as swabs, catheters, or gastro tubes for example, and a plaster to cover the surrounding area may be applied, if that is the decision of the attending physician.

Applications

The split allows a gentle encirclement around stoma entrances. The central island of viscose material will absorb and retain any secretions. The silicone coated polyurethane foam will adhere the plaster to the skin. It is non-irritant and easy to remove.

Method of use

- Clean the skin with water if necessary
- Avoid the use of alcohols, sprays or creams etc.
- Remove one side of the release film that covers the adhesive
- Position the silicone plaster using the split as a positioning aid. The silicon plaster can simply be fixed around the stoma and any tubing that has been inserted, to make a snug fit.
- Remove the other side of the release film and stick the plaster to the skin
- Secure the plaster by rubbing it gently

Ingredients:

Viscose, skin-friendly silicone gel, laminated on a flexible polyurethane foam.

Caution:

The silicone plasters are for single use only. Re-use can lead to cross-contamination. ZenSetiv silicone plasters should not be used by anyone with a known sensitivity to either silicone, polyurethane or viscose.

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