
DE Hydrokolloid Formen  
 EN Hydrocolloid Shapes  
 NL Hydrocolloïde vormen  
 SV Hydrokolloida former  
 SK hydrokoloidné tvary  
 PL hydrokoloidowe kształty

FR Formes hydrocolloïdes  
 PT Formas hidrocolóides  
 RO Forme hidrocoloid  
 EL υδροκολλοειδείς σχήματα  
 CZ hydrokoloidní tvary  
 ES formas hidrocoloides

**REF**

HSR125 HSU200 HSF311  
 HSR200 HSU311 HSF511  
 HSR090 HSU511 HSF522  
 HSR120  
 HSR522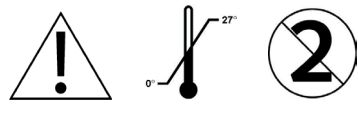

OxMed International GmbH, Groendahlscher Weg 87, 46446 Emmerich am Rhein, Germany

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**EN: ZENSETIV HYDROKOLLOID SHAPES – NON-STERILE**  
 ZenSetiv hydrocolloid shapes are made of skin friendly and absorbent hydrocolloid which is backed with a layer of flexible PE film or foam. The hydrocolloid contains polyisobutylene, carboxymethylcellulose, pectin and glycerine. ZenSetiv hydrocolloid shapes are latex-free and do not contain ingredients of animal origin.  
 ZenSetiv hydrocolloid shapes are non-sterile. They can be used in any appropriate circumstances where the attending physician or healthcare professional decides a non-sterile dressing may be applied.  
 ZenSetiv hydrocolloid shapes are available in three ranges and in different shapes and sizes, as described in the following table:

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		Variants					
		Regular		Ultra Thin		Soft Foam	
		Standard hydrocolloid with PE backing	Extra thin hydrocolloid with PE backing	Standard hydrocolloid with foam backing			
Shape	Size in cm	Article code	Artikel code	Artikel code			
Square with rounded edges	12,5 x 12,5	x	HSR125				
	20 x 20	x	HSR200	x	HSU200		
Round	9 Ø	x	HSR090				
	12 Ø	x	HSR120				
Strips with rounded edges	3 x 11			x	HSU311	x	HSF311
	5 x 11			x	HSU511	x	HSF511
	5 x 22	x	HSR522			x	HSF522

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**INDICATIONS FOR USE**

- as a skin protector with stomas, fistulas and wounds against aggressive output and against pressure from the hard components of stoma pouches, like urine taps, clamps, girdles etc.
- as fixation material to add extra adhesion to the edges of a wound dressing or the skin wafer of stoma appliances.
- The round skin wafers can be used as an extra absorption material underneath the daily stoma appliance.
- When treating psoriasis or eczema the ZenSetiv hydrocolloid shapes can be used as an occlusive dressing (variant Ultra Thin is most suitable). But please note - it is necessary to allow for enough oil or cream free areas at the edge of the hydrocolloid in order for the hydrocolloid to stick well to the skin.
- as a skin protector with heavily exudative wounds, protecting the healthy edges of the wound from maceration by exudates or by continuous contact with the saturated absorbent dressing.
- as a fixation material in allergy tests (variant Ultra Thin). The PE backing is water-resistant, therefore the user may continue to take baths and showers.

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